

## HbA1c CALIBRATOR

Lyophilized Powder

**FOR IN-VITRO DIAGNOSTICS USE ONLY**

**Pack 4x1 ml**

### INTENDED USE

HbA1c calibrators are intended to be used for calibration of HbA1c reagent. This Calibration Pack is for clinical laboratory testing use only.

After reconstitution the calibrator, no need to do lysing procedure.

### PRODUCT DESCRIPTION

The Calibrator Pack Contents:

HbA1c calibrator Level 1	1x1ml
HbA1c calibrator Level 2	1x1ml
HbA1c calibrator Level 3	1x1ml
HbA1c calibrator Level 4	1x1ml

### STORAGE & STABILITY

- The lyophilized vial should be store at 2°C to 8°C and may be used up to the expiration date on the Pack.
- Reconstituted vial are stable for 7 days at 2°C to 8°C.

### PREPARATION OF CALIBRATOR

Each lyophilized HbA1c calibrator should be reconstituted by adding 1.0 ml of distilled water. Close the vial and dissolve the contents by swirling gently to avoid formation of foam.

Allow it to stand for 30 minutes before use

### WARNING

Do not use expired Calibrator. Discard calibrator pack after use according to local regulations. Wash contaminated area with suitable disinfectant.

**Do Not Freeze.**



**HEME DIAMED LLP**

Maharashtra 401208

MFG LIC NO: MFG/IVD/2025/00073



**HEME DIAMED LLP**

2nd Floor, 4, 1981, Anjikath Lane, near  
Vallathol Junction Vazhakala, Thrikkakara,  
Edappally, Ernakulam, Kakkanaad, Kerala 682021  
MOB: +91 7395966499